





Sterling Couch Range



Therapy Couch Range



Patient Trolley Range



Seating Range









ISO 13485

ISO 14001



Kenton Road, Debenham, Suffolk IP14 6LA, United Kingdom © Copyright SEERS Medical Ltd 2023

Company Reg. № 6572698 VAT № 931596702 Printed in the United Kingdom



Tel +44 (0)1728 861 590 **Fax** +44 (0)1728 861 868

info@seersmedical.com www.seersmedical.com

> Issue № 1/23 Part № 06Z-8005-SF

Note¹: The illustrated fabric colours may vary slightly from actual fabric supplied. This occurs due to the four colour printing process. However every attempt has been made to make them as accurate as possible. Our Policy of continuous improvement means that we reserve the right to change and modify designs without prior notice. Note²: Measurements may vary +/- 2° or +/- 20mm due to manufacturing tolerances



Clinical Range

Clinical Couches for Patient Examination and Minor Treatment Procedures





Introduction

In this new brochure we cover our wide range of patient examination furniture for various medical procedures such as Gynaecology, Ultrasound, Phlebotomy, Lymphoedema and Podiatry. There is a wide choice of models and accessories to cater for various examination procedures as well as specialist treatment and minor operation techniques. Most models are variable in height with a choice of both manual or electric positioning and height operation. The development of these products has been achieved by applying our many years of experience and knowledge in this market. Much thought has been given to the design, to benefit both the comfort and security of the patients as well as providing manual handling solutions which promote good posture and ergonomics to all clinical staff when they are using this equipment. With the management and control of our own assembly line at our premises in Debenham, Suffolk in the UK, our couches comply to the CE medical devices directive 93/42/EEC, ensuring we supply high quality products every time, with short lead times to meet the tightest of deadlines. We trust you will experience this yourself when you purchase a SEERS Medical product. SEERS Medical are ISO9001 and ISO14001 accredited.

Contents

4-5	Medicare 2 Section Couch
6-7	Medicare 3 Section Couch
8-9	Medicare Mobile 2 Section Couch
10-11	Medicare Mobile 3 Section Couch
12-13	Innovation Deluxe 2 Section Couch
14-15	Innovation Deluxe 3 Section Couch
16-17	Innovation Deluxe Daycare Couch
18-19	Innovation Deluxe Dialysis Couch
20-21	Medicare Multi-Couch - Single Footrest
22-23	Medicare Podiatry Couch
24-25	Medicare Orthopaedic Couch
26-27	Medicare Phlebotomy Couch
28-29	Medicare Echocardiography Couch
30-31	Medicare Ultrasound Couch
32-33	Medicare Gynaecology Couch
34-35	Medicare Deluxe Gynaecology Couch
36-37	Medicare GP Gynaecology Couch
38-39	Medicare Bariatric 2 Section Couch
40-41	Medicare Bariatric 3 Section Couch
42-43	Medicare Bariatric Drop End Couch
44-45	Shower Trolley
46-47	Optional Accessories
48-49	Chairs, Stools & Vinyl Colour Options



Safe working load 240Kg (550lbs)



Mobile hoist clearance beneath frame



Anti-bacterial powder coated fram



Anti-microbial vinyl upholstery



Crib 5 Fire retardant upholstery



Low patient transfer height



Complies with EC Directive 93/42/EEC

Technical Information

At **SEERS** Medical we are using state of the art design tools, such as 3D solid modelling and in house rapid prototyping, to assist us in getting new designs into development and manufacture. Whilst designing new products, all manufacturing capabilities available are taken into consideration, such as CNC wood cutting, laser profiling and robot welding, which deliver reliable build quality and repeatability of accurate components, time after time. A typical innovative use of these processes is evident in a high loaded components such as the ram bracket, a common item on many **SEERS** couches, where using laser profiling we have been able to make the parts interlock, adding immense strength, even before welding! This is typical of the attention to detail that we pride ourselves with, and sets us apart from other couch manufacturers, giving us confidence in the reliability and longevity of the products we manufacture.

Mechanical Safety & Load Testing

All **SEERS** Medical products have been tested to the applicable EN 60601-1:2006+A12:2014 (E) the General requirements for basic safety and essential performance. Our couches have been thoroughly tested with a two and a half times safe working load (in excess of 600Kg loaded evenly, for in excess of one hour, to prove load support capability without failure. Our products are tested in accordance to Clauses 9.8.3.2-3 Static Forces of EN60601-1:2006 Not only do our products have high safe working loads (typically 240Kg), but our products also lift the rated safe working load from the minimum height to the maximum height smoothly.

Electrical Safety

All **SEERS** Medical products comply to EN 60601-1:2006+A12:2014 (E) the General requirements for basic safety and essential performance. In addition, the actuation and control systems used on our products are marked with the UL mark, demonstrating compliance to UL60601-1 and CAN/CSA-C22.2 No. 601.1-M90, meeting the requirements for USA and Canada. The actuation control system is also marked with approval to C-ETL-US No. 3154358.

CE Marking & Quality System


SEERS Medical operates a Quality Management System in accredited and approved to ISO 13485:2016. **SEERS** Medical products comply with the Essential Requirements in Annex I of EC Directive 93/42/EC, with all relevant design, production records, risk assessments, product testing and labelling information recorded in the documentation file labelled the "Technical File". All products carry a unique serial number for trace-ability of build date, model number and associated build records.

Flammability


The upholstered couch sections satisfies flammability requirements to the standards of BS 7176:1995 (medium hazard) using ignition sources 0 (cigarette), 1 (gas) as defined in BS EN. 1921-1 and 2 and 5 (crib) BS 5852:1990.

Warranty

The variable height couches detailed in this brochure have a lifetime frame warranty and five years warranty on all other components.

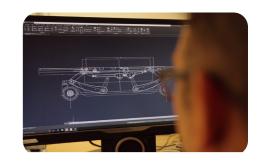

CAD design technology

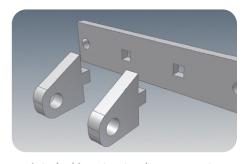


Interlocking structural components




Laser profiling

 $\mathbf{2}$



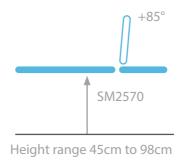

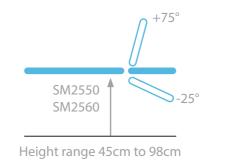



Medicare 2 Section Couch







A very strong and rigid couch available with a wide range of accessories. Suitable for supporting very heavy patients for examination and minor treatment procedures.



Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Gas assisted backrest adjustable from -25° to 75° on SM2550 and SM2560 and incorporates anti-trap design
- Electric backrest adjustable from 0° to 85° on SM2570
- Choice of hydraulic or electric height elevation
- Excellent height range from 45cm to 98cm
- Retractable wheel design with adjustable foot for stability
- Purpose made precision bearings on all pivot point
- Clearance beneath frame for mobile hoist
- Hand switch fitted as standard on electric models
- Frames pre-drilled for addition of accessories
- Anti-bacterial powder coated frame
- 63cm wide Easy Clean Upholstery as standard
- Choice of Standard or Standard+ upholstery colours

Optional Accessories

- **9002** Side support rails (Pair)
- **9002P** Side support cushions (Pair)
- **6004** Breathing hole and plug
- **6005** Paper roll holder
- 5006 Upper limb support
- **6007** Removable head/knee support
- **6053** Protective base frame cover
- **6054** Telescopic IV pole
- **057** Battery back-up facility
- **-EPFS** Perimeter Foot Switch for SM2560
- **WF** Single foot switch for SM2560
- **WF** Double foot switch for SM2570
- 9200/70 70cm Width Easy Clean Upholstery






Low patient transfer height

Negative head section to -25°





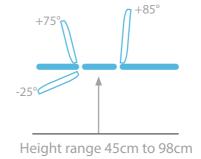



Standard Dimensions





*When fitted with 9300/70 Easy Clean Upholstery

Medicare 3 Section Couch









The Medicare 3 Section Couch provides a very stable and versatile support surface for prone, supine and seated treatment procedures and examinations. Both the elevating sections can be set to a wide range of angles for specific patient positioning.

Models

SM3550 Hydraulic with gas assisted backrest SM3560 Electric with gas assisted backrest SM3570



Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)

- Choice of hydraulic or electric height elevation

- Purpose made precision bearings on all pivot points
- Clearance beneath frame for mobile hoist
- Hand switch fitted as standard on electric models
- Frames pre-drilled for addition of accessories
- 63cm wide Easy Clean Upholstery as standard
- Choice of Standard or Standard+ upholstery colours

Optional Accessories

9003 Side support rails (Pair)

9002P Side support cushions (Pair)

Breathing hole and plug

Paper roll holder

Upper limb support

Removable head/knee support

Protective base frame cover

Telescopic IV pole

6057 Battery back-up facility

Perimeter Foot Switch for SM3560

Single foot switch for SM3560

Double foot switch for SM3570

9300/70 70cm Width Easy Clean Upholstery




Negative head/ foot section to -25°









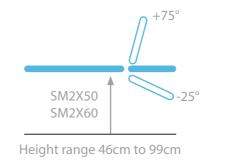



Medicare Mobile 2 Section Couch







Very strong and rigid 2 section mobile treatment couches supplied with side support rails and a paper roll holder. The large wheel system incorporates a steering facility and the couches have excellent height ranges with a low patient transfer height of just 46cm.

Models

Independently Braking Castors (IBC)

SM2450 Hydraulic with gas assisted backres
 SM2460 Electric with gas assisted backrest
 SM2470 Electric with electric backrest

Central Locking Castors (LMWD)

SM2650 Hydraulic with gas assisted backrest
 SM2660 Electric with gas assisted backrest
 SM2670 Electric with electric backrest

Model SM2650

Shown with optional 6053 Base Cover, 9002P Side Cushions, 6054 IV Pole and 6007 Head Cushion

Colour Sky Blue

Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Gas assisted backrest adjustable from -25° to 75° on SM2450, SM2460, SM2650 and SM2660
- Electric backrest adjustable from 0° to 85° on SM2470/267
- Gas assisted backrest incorporates anti-trap design
- Choice of hydraulic or electric height elevation
- Excellent height range from 46cm to 99cm
- Side Rails & Paper Roll Holder fitted as standard
- Choice of independently or centrally locking castors
- Purpose made precision bearings on all pivot points
- Clearance beneath frame for mobile hoist
- Hand switch fitted as standard on electric models
- Frames pre-drilled for addition of accessories
- Anti-bacterial powder coated frame
- 63cm wide Easy Clean Upholstery as standard
- Choice of Standard or Standard+ upholstery colours

Optional Accessories

9002P Side support cushions (Pair)

6004 Breathing hole and plug

6006 Upper limb support

6007 Removable head/knee support

6053 Protective base frame cover

6054 Telescopic IV pole

6057 Battery back-up facility

-1WF Single foot switch for SM2X60

-2WF Double foot switch for SM2X70

9200/70 70cm Width Easy Clean Upholstery








Seated position Shown with optional 6053 Base Cover and 6006 Upper Limb Support





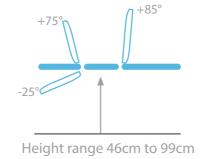





Medicare Mobile 3 Section Couch Low Transfer Height









Very strong and rigid 3 section mobile treatment couches supplied with side support rails and a paper roll holder as standard. The Large Mobile Wheel Design incorporates central locking castors & steering facility, and the couches have excellent height ranges with a low patient transfer height.

Models

Independently Braking Castors (IBC)

SM3450 Hydraulic with gas assisted backres
 SM3460 Electric with gas assisted backrest
 SM3470 Electric with electric backrest

Central Locking Castors (LMWD)

SM3650 Hydraulic with gas assisted backrest
SM3660 Electric with gas assisted backrest
SM3670 Electric with electric backrest

Model SM3650 Colour Sky Blue

Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Gas assisted or electric backrest adjustable from 0° to +85°
- Gas assisted head section adjustable from -25° to +75°
- Gas assisted sections incorporate anti-trap design
- Choice of hydraulic or electric height elevation
- Excellent height range from 46cm to 99cm
- Choice of independently or centrally locking castors
- Side Rails & Paper Roll Holder fitted as standard
- Purpose made precision bearings on all pivot points
- Clearance beneath frame for mobile hoist
- Hand switch fitted as standard on electric models
- Frames pre-drilled for addition of accessorie
- Anti-bacterial powder coated frame
- 63cm wide Easy Clean Upholstery as standard
- Choice of Standard or Standard+ upholstery colours

Optional Accessories


- **9002P** Side support cushions (Pair)
- **6004** Breathing hole and plug
- **6006** Upper limb support
- **6007** Removable head/knee support
- **6053** Protective base frame cover
- **6054** Telescopic IV pole
- **6057** Battery back-up facility
- -**1WF** Single foot switch for SM3X60
- **-2WF** Double foot switch for SM3X70
- 9300/70 70cm Width Easy Clean Upholstery



Low patient transfer height





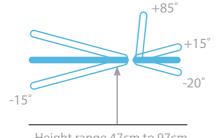



Height range 47cm to 97cm

Innovation Deluxe 2 Section









The Innovation Deluxe 2 section couch is suitable for use within specialist critical care environments, endoscopy, ultrasound scanning and minor treatment procedures. Electric height, backrest and 2-way tilt ensure smooth transition when positioning patients for specialist procedures and treatments. Auto-electric CPR functionality and auto-levelling software return the couch quickly to a fully horizontal resuscitation position in the event of an emergency.

Models

MG2475

12

Electric with electric backrest and 2-way

tilt. Large independently locking

wheel design

MG2675 Electric with electric backrest and 2-way

tilt. Large central locking wheel design

with steering facility

Model MG2675 with large central locking wheel design with steering facility



- Electric height range from 47cm to 97cm
- Electric trendelenburg tilt to -/+15°
- Safe working load and lifting capacity of 250Kg (550lbs)
- Auto-electric CPR function via handset
- Auto-intelligent operation
- Vertical lifting design to minimise space and provide maximum stability throughout all profiling positions
- Smooth moulded base cover minimises dirt traps and facilitates easy cleaning
- Large TENTE® Ø125mm castors
- **SEERS** ComfortPlus[™] Upholstery features anti-microbial vinyl incorporating pressure relieving design for additional patient comfort with a foam depth of 60mm
- 70cm wide upholstery
- Clearance beneath frame for mobile hoist
- Head Cushion & Paper Roll Holder fitted as standard
- Choice of Standard or Standard+ upholstery colours

Optional Accessories

6003 Side support rails (Pair)

9002P Side support rail cushions (Pair)

Breathing hole and plug Upper limb support

6054 Telescopic IV pole **6057** Battery back-up facility






Trendelenburg tilt to -15°





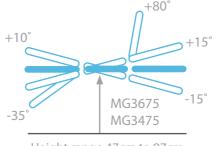







Height range 47cm to 97cm

Innovation Deluxe 3 Section







Anti Bacterial



The Innovation Deluxe 3 section couch is designed for use within specialist critical care environments. Electric height, backrest and foot section profiling and 2-way tilt ensure a smooth transition when positioning patients for specialist procedures and treatments. Auto-electric CPR functionality and auto levelling software return the couch quickly to a fully horizontal resuscitation position in the event of an emergency.

Models

MG3475 Electric profiling of height, backrest and

foot section with auto-levelling, 2-way tilt and CPR facility. Large independently

MG3675 Electric profiling of height, backrest and

> tilt and CPR facility. Large central locking wheel design with steering facility

> > Model MG3475 with large independently braked wheel design

Standard Features

- Electric trendelenburg tilt to -/+15°

- Safe working load and lifting capacity of 250Kg (550lbs)
- Auto-electric CPR function via handset
- Auto-intelligent operation
- Vertical lifting design to minimise space and provide maximum stability throughout all profiling positions
- Smooth moulded base cover minimises dirt traps and facilitates easy cleaning
- Large TENTE® Ø125mm castors
- **SEERS** ComfortPlus™ Upholstery features anti-microbial vinyl incorporating pressure relieving design for additional patient comfort with a foam depth of 60mm
- Clearance beneath frame for mobile hoist
- Head Cushion & Paper Roll Holder fitted as standard
- Choice of Standard or Standard+ upholstery colours

Optional Accessories

6003 Side support rails (Pair)

9002P Side support rail cushions (Pair)

6004 Breathing hole and plug for prone

treatments

6006 Upper limb support Telescopic IV pole

6057 Battery back-up facility





Low patient transfer height






Reverse trendelenburg tilt to +15°





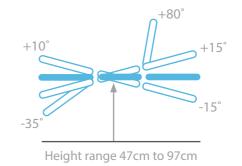



Innovation Deluxe Daycare









The Innovation Deluxe Daycare couch is designed for specialist medical procedures, critical care and oncology units. The fully electric functionality provides smooth lifting and patient profiling during patient examination and treatment. The couch features 2-way tilt and auto-electric CPR facility with auto-levelling software.

Models

MG3493 Electric profiling of height, backrest and

foot section with auto-levelling, 2-way tilt and CPR facility. Large independently

braked wheel design

MG3693 Electric profiling of height, backrest and

foot section with auto-levelling, 2-way tilt and CPR facility. Large central locking wheel design with steering facility



2-way trendelenburg tilt facility to +/-15°

Standard Features

- Electric height range from 47cm to 97cm
- Electric trendelenburg tilt to -/+15°
- Electric backrest adjustable from 0° to +8
- Flectric footrest adjustable from +10° to -35
- Safe working load and lifting capacity of 250Kg (550lbs)
- Auto-electric CPR function via handset
- Auto-intelligent operation
- Vertical lifting design to minimise space and provide maximum stability throughout all profiling positions
- Smooth moulded base cover minimises dirt traps and facilitates easy cleaning
- Anti-bacterial powder coated frame and base cover
- Large TENTE® Ø125mm castors
- **SEERS** ComfortPlus™ Upholstery features anti-microbial vinyl incorporating pressure relieving design for additional patient comfort with a foam depth of 60mm
- 70cm standard width upholstery
- Clearance beneath frame for mobile hoist
- Head Cushion, Paper Roll Holder, Side Rails and Side Support Cushions fitted as standard
- Fold down side rails and cushions provide patient comfort and security
- Choice of Standard and Standard+ upholstery colours

Optional Accessories

6054 Telescopic IV pole

6057 Battery back-up facility

6241 Patient lap tray

6242 Adjustable foot board



Model 6241 optional patient lap tray



Model 6242 optional adjustable foot board




Low patient transfer height

Available Upholstery Colours:



Model MG3693 with large central locking wheel design with steering facility



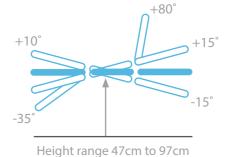



Innovation Deluxe Dialysis





Anti Bacterial





The Innovation Deluxe Dialysis couch has been designed for use within specialist renal dialysis environments. The couch features auto-electric CPR and fully electric profiling with a 2-way tilt facility to ensure a smooth transition when positioning patients for dialysis procedures and treatments.

Models

MG3490 Electric profiling of height, backrest and

and CPR facility. Large independently

braked wheel design

Electric profiling of height, backrest and MG3690

> tilt and CPR facility. Large central locking wheel design with steering facility



Standard Features

- Electric trendelenburg tilt to -/+15°

- Safe working load and lifting capacity of 250Kg (550lbs)
- Auto-electric CPR function via handset
- Auto-intelligent operation
- Vertical lifting design to minimise space and provide maximum stability throughout all profiling positions
- Smooth moulded base cover minimises dirt traps and facilitates easy cleaning
- Large TENTE® Ø125mm castors
- **SEERS** ComfortPlus™ Upholstery features anti-microbial vinyl incorporating pressure relieving design for additional patient comfort with a foam depth of 60mm
- Clearance beneath frame for mobile hoist
- Head Cushion & Paper Roll Holder fitted as standard
- Multi-adjustable arm rests and adjustable upholstered foot board included for patient comfort
- Choice of Standard and Standard+ upholstery colours

Optional Accessories 6054 Telescopic IV pole

Battery back-up facility

6241 Patient lap tray



Low patient transfer height




2-way trendelenburg tilt facility to +/-15°



Fully profiled for patient comfort





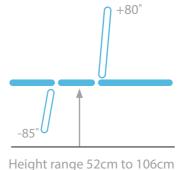



Medicare Multi-Couch







Incorporating both an adjustable angle backrest and a single drop end foot section, these models are suitable for a wide range of seated and supine patient examination procedures. The slightly narrow foot section enables the addition of leg supports or stirrups for gynaecological procedures.

Models

SM9556 Electric with gas assisted backrest SM9566 SM9576 Electric with electric backrest



Model 6045 optional leg supports for gynaecology



Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Gas assisted foot section adjustable from 0° to -85°
- Gas assisted sections incorporate an anti-trap design
- Choice of hydraulic or electric height elevation
- Compact frame design with retractable wheel system
- Adjustable foot for stability on uneven surfaces
- Anti-bacterial powder coated frame
- Clearance beneath frame for mobile hoist
- 65cm standard width upholstery
- Choice of Standard and Standard+ upholstery colours

Optional Accessories

6004 Breathing hole and plug

Paper roll holder

Upper limb support

Removable head/knee cushion

Lithotomy stirrups

Leg supports for gynaecology

Protective base frame cover

Telescopic IV pole

Battery back-up facility

Single foot switch for SM9566

Double foot switch for SM9576





lithotomy stirrups


Low patient transfer height


Available Upholstery Colours:

20





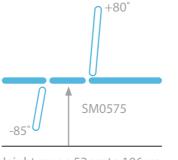

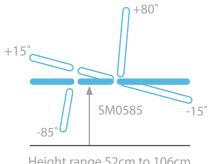



Height range 52cm to 106cm

Height range 52cm to 106cm

Medicare Podiatry Couch









Incorporating both an adjustable angle backrest with patient armrests and dual adjustable footrests, these models are suitable for a wide range of seated examination procedures such as podiatry, leg ulcer clinics, orthopaedic casting (plaster room), as well as being ideal for GP surgeries and health centres.

Models

SM0575 SM0585 Electric height and backrest Electric height, tilt and backrest



6004 Breathing hole and plug for prone treatments



Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Electric backrest adjustable from 0° to +80°
- Gas assisted foot sections adjustable from 0° to -85° and incorporate anti-trap design gas struts
- Footrests extend for tall patients
- Fold-down armrests have angular adjustment
- Electric height elevation via hand control
- Excellent height range from 52cm to 106cm
- SM0585 includes paper roll holder and matching headrest as standard
- Electric tilt facility of -15° on SM0585
- Compact frame design with retractable wheel system
- Adjustable foot for stability on uneven surfaces
- Purpose made precision bearings on all pivot points
- Anti-bacterial powder coated frame
- Clearance beneath frame for mobile hoist
- 65cm standard width upholstery
- Choice of Standard or Standard+ unholstery colour

Optional Accessories

- **6052** Protective base frame cover
 - **O7** Head Cushion (Included on SM0585)
- **6005** Paper roll holder (Included on SM0585)
- **6004** Breathing hole and plug
- 6054 Telescopic IV Pole Attachment
- **6057** Battery back-up facility
- **2WF** Double foot switch for SM0575
- **-3WF** Triple foot switch for SM0585





Matching headrest as standard on Model SM0585. Option on Model SM0575








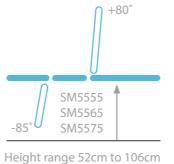

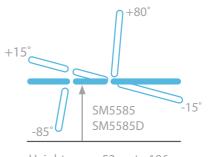

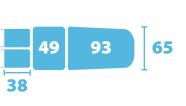

Height range 52cm to 106cm

Medicare Orthopaedic Couch









Designed specifically for plaster casting technicians, this couch features individually adjustable leg sections and is available with a range of optional accessories to assist and support the technician in carrying out various casting procedures. The Deluxe version is supplied as a package with base shroud, set of accessories and matching stool.

Models

SM5555 Hydraulic with gas assisted backrest SM5565 SM5575 Electric with electric backrest SM5585 Electric with electric height, tilt and backrest

SM5585D Deluxe model with stool, base cover and

Orthopaedic accessories set

Model SM5565 Colour Dark Blue



5007 Orthopaedic Full Leg Seat Module



Orthopaedic Arm Support



Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Gas assisted or electric backrest adjustable from 0° to +80°
- Gas assisted foot sections adjustable from 0° to -85°
- Trendelenburg tilt to -15° on Models SM5585/ SM5585D
- Gas assisted sections incorporate anti-trap design
- Footrests extend for tall patients
- Choice of hydraulic or electric height elevation
- Excellent height range from 52cm to 106cm
- Compact frame design with retractable wheel system
- Adjustable foot for stability on uneven surfaces
- Purpose made precision bearings on all pivot points
- Anti-bacterial powder coated frame
- Clearance beneath frame for mobile hoist
- Hand switch fitted as standard on electric models
- Choice of Standard and Standard+ upholstery colours

Optional Accessories

- **5006** Orthopaedic arm support
- Orthopaedic full leg seat module
- Knee support roll set
- Orthopaedic leg support attachment
- Breathing hole and plug
- **6005** Paper roll holder
- Removable head/knee cushion
- **6052** Protective base frame cover
- Battery back-up facility
- Single foot switch for SM5565
- Double foot switch for SM5575
- Triple foot switch for SM5585/SM5585D



Orthopaedic Leg Support Attachment




Knee Support Roll Set



extended patient footrest





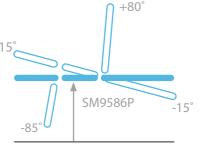



Height range 52cm to 106cm

Height range 52cm to 106cm

Medicare Phlebotomy Couch










A specialist couch designed to provide a comfortable and easy to mount seat position for blood sampling procedures. All models are fitted with a pair of multi-adjustable arm rests and side support loops for the stability and safety of the patient when mounting and dismounting the couch. The seat section is inclined for additional patient security.

Models

SM9556P SM9566P SM9576P SM9586P Electric height, tilt and backrest

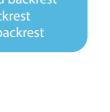



Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Electric Trendelenburg tilt to -15° on SM9586P
- Patient hand-grip supports to aid mount/dismount from the couch as well, as providing patient security
- Fully adjustable arm supports for use on either side of couch
- Paper roll holder fitted as standard
- Choice of hydraulic or electric height elevation
- Excellent height range from 52cm to 106cm

- Anti-bacterial powder coated frame
- Inclined seat section encourages patients to sit back
- Clearance beneath frame for mobile hoist

Optional Accessories

6052 Protective base frame cover

Battery back-up facility

Head cushion (Included on SM9586P)

Single foot switch for SM9566P

Double foot switch for SM9576P Triple foot switch for SM9586P

Low Patient Transfer Height

















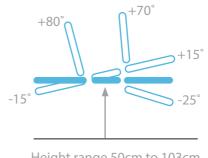

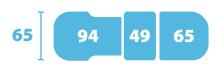

Height range 50cm to 103cm

Medicare Echocardiography Couch









Designed specifically for Echocardiography, this couch has electric operation of height, tilt and backrest angle. The backrest has a cut-away design for improved application of the transducer during scanning procedures and a specially designed stool for the clinician is also provided. The couch is also supplied with an integral footrest for the clinician, as well as a patient hand-grip and head cushion for the patients comfort and security.

Models

SM3590

Electric height, tilt and backrest. Electric height, tilt and backrest.

SM3690 Central locking wheel system



Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Electric height, tilt and backrest adjustment

- Electric trendelenburg tilt to -15°
- incorporating anti-trap design
- Ergonomic backrest designed for improved transducer positioning
- Integral footrest for clinician when traditional right-hand scanning, together with specially designed stool
- Hand-grip and side support with cushion for patient positioning
- Purpose made precision bearings on all pivot points
- Anti-bacterial powder coated frame
- Paper Roll Holder and Head Support Cushion fitted as standard
- Retractable wheel design with adjustable foot for stability on
- Choice of Standard and Standard+ upholstery colours

Optional Accessories

-3WF Triple foot switch operation Battery back-up facility

6123 Fully adjustable arm support for

the clinician



Special design stool provided for clinician, included as standard






Model 6123 optional adjustable arm support for clinician





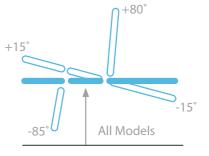



Height range 52cm to 106cm

Medicare Ultrasound Couch









Providing smooth electric positioning of height, tilt and backrest angle, these couches feature a compact base frame and are ideal for both supine and seated procedures. The deluxe model includes the base frame cover and matching operators chair. The narrower foot section enables the fitting of optional leg supports or stirrups for gynaecological procedures.

Models

SM9686 Electric height, tilt and backrest, gas assisted foot section

SM9686D Deluxe Model with Base Cover & Operators

Chair

SM9696 Electric height, tilt, backrest and foot sectionSM9696D Deluxe Model with Base Cover & Operators

Chair



Model 6044 optional lithotomy stirrups



Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Electric adjustment of height, tilt, backrest and foot section
- Electric backrest adjustable from 0° to +80°
- Adjustable angle footrest from 0° to -85°
- Electric trendelenburg angle to -15°
- Choice of gas assisted or electric operated foot section
- Excellent height range from 52cm to 106cm
- Paper Roll Holder and Head Cushion fitted as standard
- Compact frame design with retractable wheel system
- Adjustable foot for stability on uneven surfaces
- Purpose made precision bearings on all pivot points
- Anti-bacterial powder coated frame
- Clearance beneath frame for mobile hoist
- 70cm standard width upholstery
- Choice of Standard and Standard+ upholstery colours

Optional Accessories

6004 Breathing hole and plug

6041 Side support loops

6044 Lithotomy Stirrups

6045 Gynaecology Leg Supports

Protective base frame cover

6054 Telescopic IV pole

6057 Battery back-up facility

-3WF Triple foot switch for SM9686/SM9686D

-4WF Quad foot switch for SM9696/SM9696D













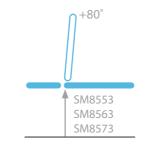

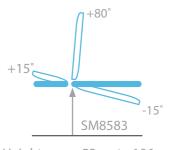



Height range 52cm to 106cm

Height range 52cm to 106cm

Medicare Gynaecology Couch









Designed for general gynaecology procedures, these models are available with either hydraulic or electric height operation and feature a wide choice of optional accessories to meet both financial and clinical requirements. All models feature a crescent shaped seat design and are supplied with fully adjustable leg supports, paper roll holder, side support loops and debris tray for gynaecology procedures.

Models

SM8553 Hydraulic with gas assisted backrest SM8563 Electric with gas assisted backrest SM8573 Electric height and backrest SM8583 Electric height, tilt and backrest



6043 Removable Leg Extension



6051 Fold-Down Leg Extension



Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Crescent shaped seat design
- Fully adjustable patient leg supports
- Fitted with debris tray, Side Loops & Paper Roll Holder
- Choice of hydraulic or electric height elevation
- Excellent height range from 52cm to 106cm
- Adjustable foot for stability on uneven floors
- Purpose made precision bearings on all pivot points
- Anti-bacterial powder coated frame
- Electric trendelenburg tilt to -15° on model SM8583
- 65cm standard width upholstery
- Choice of Standard and Standard+ upholstery colours



ST6100 **Operators Chair**

Optional Accessories

6007 Head Cushion (Included on SM8583)

Removable leg extension accessory

Fold-down leg extension accessory

Protective base frame cover Battery back-up facility

Single foot switch for SM8563

Double foot switch for SM8573


Triple foot switch for SM8583





Model SM8563 Colour light grey





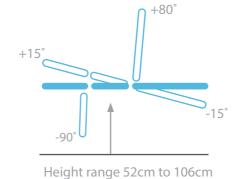

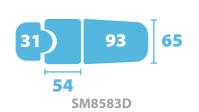

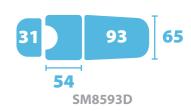

Medicare Deluxe Gynaecology Couch









Designed for colposcopy and general gynaecology procedures, these Deluxe models feature electric positioning for height, backrest angle and tilt position. They are supplied with a special set of accessories which include a matching operator's chair, base frame cover, a choice of leg extension, leg supports, matching head support cushion, side support loops, debris tray and paper roll holder.



SM8583D

SM8593D



Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Electric control of height, backrest and tilt angle
- Excellent height range from 52cm to 106cm
- Crescent shaped seat design
- Fully adjustable patient leg supports
- Fitted with debris tray, Side Support Loops and Paper Roll Holder
- Supplied with operators chair, head cushion and base frame cover
- Choice of Removable or Fold-Down Leg Extension

- Anti-bacterial powder coated frame
- Hand switch fitted as standard
- Choice of Standard and Standard+ upholstery colours



Fold-down leg extension


Optional Accessories


-3WF Triple foot switch **6057** Battery back-up facility







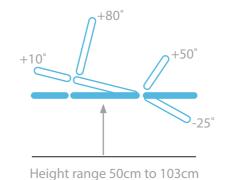

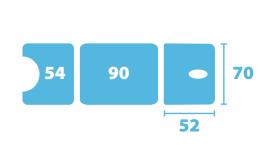

Optional Accessories

Gynaecology leg supports

Protective base frame cover Battery back-up facility

6007 Head support cushion

Triple foot switch

Medicare GP Gynaecology Couch









A multi-purpose couch providing electric positioning for gynaecology procedures, as well as a full length couch surface for most general prone and supine examinations and minor treatment procedures.

Models

SM8543 Electric height, tilt and backrest with

manual head section, retractable

SM8543D

cover, matching operators chair and head support cushion, retractable



ST6100 **Operators Chair**



Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Electric adjustable angle backrest from horizontal to +80°
- Electrically adjustable seat angle with trendelenburg to -10°
- Breathing hole and plug fitted as standard

- Clearance beneath frame for mobile hoist
- Anti-bacterial powder coated frame
- Supplied complete with lithotomy stirrups and debris tray
- Paper roll holder fitted as standard
- Choice of Standard and Standard+ upholstery colours







Available Upholstery Colours:



Model SM8543 Colour Black



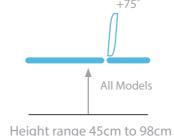

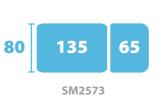

SM2673

SM4542 SM4642

Height range 45cm to 98cm

Medicare Bariatric 2 Section Couch









The Bariatric 2 Section Couches feature electric height and backrest operation for effortless patient positioning for various medical examinations and treatment procedures. There is a choice of widths available - Models SM2573 and SM2673 are available with a width of 80cm and Models SM4542 and SM4642 which have a wide frame, are available at 105cm as standard. All couches provide a safe working load and lifting capacity of 325Kg (715lbs).

Standard Features

- Safe working load and lifting capacity of 325Kg (715lbs)
- Choice of retractable wheels or central locking castors
- Electric backrest adjustable from 0° to +75°

- Anti-bacterial powder coated frame
- Clearance beneath frame for mobile hoist
- Choice of 80cm or 105cm width upholstery
- Hand switch fitted as standard
- Choice of Standard and Standard+ upholstery colours

Optional Accessories

6004 Breathing hole and plug

Paper roll holder

6074 Side support rail (Pair)

9002P Side support rail cushions (Pair) **6076** Memory foam upholstery (60mm)

Double foot switch

Models

SM2573

SM2673 Electric with electric backrest 80cm wide. Central Locking Castors

SM4542

SM4642

Central Locking Castors






Model SM4542







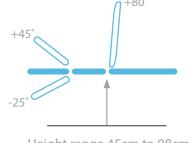



Height range 45cm to 98cm

Medicare Bariatric 3 Section Couch







Anti Bacterial



The Bariatric 3 Section Couch features electric height, backrest and foot section operation for effortless patient positioning for a wide range of examination procedures. There is a choice of wheel system available and the couch provides a safe working load and lifting capacity of 325Kg (715lbs).

Models

SM3673

SM3573 section. Retractable wheel system

Electric with electric backrest and foot

section. Central locking wheel system



Standard Features

- Safe working load and lifting capacity of 325Kg (715lbs)
- Choice of retractable wheel or large wheel design, with central locking & steering facility

- Purpose made precision bearings on all pivot points
- Clearance beneath frame for mobile hoist

Low patient transfer height



Optional Accessories

6005 Paper roll holder

6007 Head support cushion

Side support rails (Pair)

9002P Side Cushions (Pair)

6076 Memory foam upholstery (60mm)


-3WF Triple Foot Switch





Model SM3573 Colour Light Grey





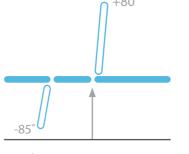

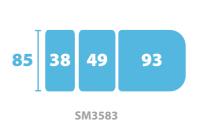

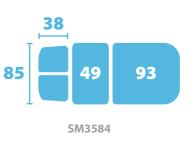

Height range 50cm to 105cm

Medicare Bariatric Drop End Couch









The Bariatric Drop End Couches feature electric independently operated foot sections for effortless positioning for various lower limb procedures. The electric backrest supports the patient in the sitting position. There is a choice of models with either single or dual foot sections, all with a safe working load and lifting capacity of 325Kg (715lbs).



SM3583

Electric height, backrest and single foot

SM3584



Standard Features

- Safe working load and lifting capacity of 325Kg (715lbs)
- Retractable wheel design with adjustable foot for stability

- Purpose made precision bearings on all pivot points
- Clearance beneath frame for mobile hoist
- Hand switch fitted as standard



Dual Adjustable Electric Footrests on SM3584

Optional Accessories

6005 Paper roll holder

Head support cushion

Side support loops (Pair)

Side support rails (Pair)


9002P Side support rail cushions (Pair)

Triple foot switch for SM3583 Quad foot switch for SM3584







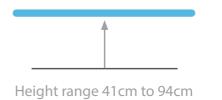



Standard

Shower Trolley



Designed to provide transport to and from the showering area, the Shower Trolley features a foam padded PVC liner for added patient comfort, a linked wheel system with central locking and steering facility, as well as fold down side support rails for patient transfer. The trolley is available with a rechargeable battery option as an alternative to the hydraulic height operation.

Models

ST7700 Hydraulic

ST7800 Electric with Rechargeable

Battery

Model ST7800 Electric model with rechargeable battery shown at mid height



d PVC liner

Standard Features

- Safe working load and lifting capacity of 240Kg (528lbs)
- Large wheel design, with central locking and steering facility
- Hydraulic height adjustment 41cm to 94cm via foot pedal
 Delawarahla hattamana dala 41cm to 94cm via hand aviita
- Rechargeable battery models 41cm to 94cm via hand switch
- Battery operated models include a spare battery pack and charging unit
- Foam padded PVC liner with side supports and end panels for comfort and security
- Supplied with foam padded PVC patient support cushion
- The side support rails fold down to assist patient transfer
- Low gradient tilt towards foot end for water drainage via integral waste outlet
- Plug and drain hose supplied complete with storage bracket
- Bumpers fitted to all corners for protection during transport procedures
- Bumpers can be removed to reduce the width to 65cm to access tight doorways
- Lowers to wheelchair height for ease of patient transfer
- Purpose made precision bearings on all pivot points
- Clearance beneath frame for mobile hoist

Optional Accessories

6093 Replacement PVC liner

Replacement Patient PVC Cushion

6092 Replacement Battery Pack

Model ST7700 Standard hydraulic model at low height for patient transfer





 $\Delta\Delta$



Optional Accessories



-1WF

Single foot switch option (alternative to standard single hand switch).



-2WF

Double foot switch option (alternative to standard double hand switch).



-3WF

Triple foot switch option (alternative to standard triple hand switch)



-EFPS

The perimeter foot switch is a "hands free" foot operated system for height adjustment. Medicare 2 & 3 sections only.



6057

Battery back-up available for most electric operated models.



6052 & 6053

An easy to clean protective base frame cover that conceals the moving parts of the framework



6002 & 6003

Fold-down side support rails for 2 & 3 section couches, providing patient support and security.



9002P

Upholstered cushions can be added to the side support rails for additional comfort, support and protection.

If you have any queries on any of the optional accessories listed above please contact us for further details. Please note some of the accessories are supplied as standard with specific couches so please ensure you check before purchasing your chosen product as all other options come at an additional cost.



6041

Side support loops provide patient support and security and are easily removable.



6054

Telescopic IV pole has fixing points on both sides of the couch and is easily adjusted to the required height



6005

The paper roll holder provides a protective barrier between the patient and the upholstered surface.



6004

Breathing hole & plug for improved patient comfort for prone treatments.



6007

The head support cushion is easily attached to the couch and can also be used as a support for lower limbs.



6006

The upper limb support has multi-angular adjustment and can be fitted to either side of the couch.



5008

The knee roll support set is an ideal accessory for supporting limbs during medical procedures.



6020

A light weight free-standing adjustable leg support for lower limb procedures. Supplied with carrying case.



6044

Traditional lithotomy stirrups for gynaecology procedures.



6045

Leg supports for gynaecology procedures. Fully adjustable for patient comfort and security.

46 47











Chairs & Stools



MC6158

Surgeons / Sonographers chair, standard model. Heigh range of 45-59cm. Polished aluminium gas strut, five star base and castors.



ST6100

Standard operators chair with 2 lever operation for height and backrest angle. Height range of 48-60cm. Black five star base and castors.



ST6103

Standard operators stool with all round release lever for height adjustment with range of 48-60cm. Black five star base and castors.



ST6144

Economy saddle stool with single lever height adjustment with a range of 48-60cm. Black five star base and castors.



MC6178

Round general medical chair, standard model. Height range of 45-59cm. Polished aluminium gas strut, five star base and castors.



ST6101

High operators chair with 2 lever operation for height and backrest angle. Height range of 53-73cm. Adjustable foot support ring. Black five star base and glides.



ST6104

High operators stool with all round release lever for height adjustment with range of 53-73cm. Adjustable foot support ring. Black five star base and glides.



MC6107

lever operation for height and seat tilt facility. Height range of 54-71cm. Polished aluminium five star base and castors.

Note¹: See Medical Seating brochure for more options.

Mundial Standard Range

A range of globally popular, hard-wearing vinyl colours that will retain performance for many years. Abrasion Resistance to 150,000 cycles. Available as standard on all upholstered couches and seating products.



Mundial Standard+ Range*

A selection of unique coloured, hard-wearing vinyl colours that will retain performance for many years. Abrasion Resistance to 150,000 cycles.

* Available at an additional cost for all Clinical Range Couches, excluding Shower Trolleys.



Other colours are available on request

Please Note: The illustrated vinyl colours may vary slightly from actual vinyl supplied. This occurs due to the four colour printing process, however every attempt has been made to make them as accurate as possible.

48